

## RegenerOss® CC Cortico-Cancellous Particulate

RegenerOss® CC Cortico-Cancellous Particulate is an anatomic-based mix of cortical and cancellous particulate that can be used to fill bony voids in a variety of dental applications. The graft has been sterilized using the Cancellite SP® Sterilization Process, lyophilized, and terminally sterilized.



# The Natural Choice for Bone Grafting.

### Natural Blend

- Graft has a natural donor-based mix of cortical and cancellous particles
- Sourced from a single donor
- Available in two particle size ranges

### Validated Sterilization Process

- The proprietary Cancellite SP® Sterilization Process inactivates and/or removes bacteria, viruses, fungi, and spores without the use of antibiotics
- Designed to preserve biological integrity and natural collagen structure of bone
- Scientifically proven and clinically successful\*
- Low-dose gamma irradiation is applied terminally

### Clinical Applications

- Extraction socket preservation
- Ridge augmentation and reconstruction
- Periodontal and peri-implant defects
- Sinus floor elevation

\* Data on file at ZimVie Dental.

# Cancelle SP®

## Sterilization Process

The Cancelle SP® Sterilization Process is a proprietary method for sterilizing and preserving the graft. Through a combination of oxidative treatments and alcohol washes, pathogens are removed and/or inactivated and debris is removed. No antibiotics are used in the process.


Cleansing rinses remove residual chemicals and low-dose gamma irradiation is applied terminally to achieve a sterility assurance level (SAL) of 10<sup>-6</sup>. The process preserves the utility and biocompatibility of the graft.

### PROPRIETARY GRAFT STERILIZATION PROCESS

- Oxidative Treatment
- Alcohol Treatment
- Rinsing
- Lyophilization
- Gamma Irradiation

### RegenerOss CC Cortico-Cancellous Particulate

- The ratio of cortical and cancellous bone in the graft is determined by the natural anatomy of the donor.

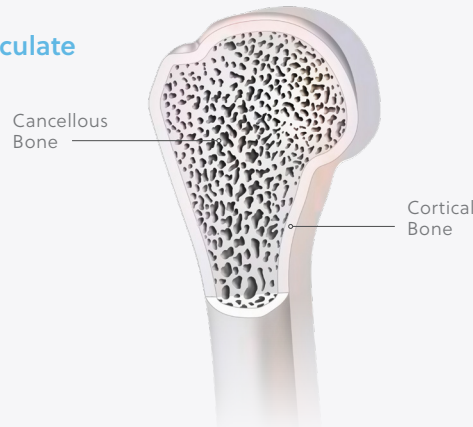

### Ordering Information

SKU	Description	Volume / Size	SKU	Description	Volume / Size
RMCCS050	RegenerOss CC Cortico-Cancellous Particulate-Small	0.5cc, 0.125 - 1.0mm	RMCCCL050	RegenerOss CC Cortico-Cancellous Particulate-Large	0.5cc, 1.0 - 2.0mm
RMCCS100		1.0cc, 0.125 - 1.0mm	RMCCCL100		1.0cc, 1.0 - 2.0mm
RMCCS200		2.0cc, 0.125 - 1.0mm	RMCCCL200		2.0cc, 1.0 - 2.0mm



Scan for IFU

For more information, visit [ZimVie.com](http://ZimVie.com)

**ZimVie**  
 4555 Riverside Drive  
 Palm Beach Gardens, FL 33410  
 1-800-342-5454  
 Phone: +1-561-776-6700  
 Fax: +1-561-776-1272



Unless otherwise indicated, as referenced herein, all trademarks are the property of ZimVie; and all products are manufactured by one or more of the dental subsidiaries of ZimVie (Biomet 3i, LLC, Zimmer Dental Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. RegenerOss CC Cortico-Cancellous Particulate is processed by RTI Surgical, Inc. Cancelle SP® is a registered trademark of RTI Surgical, Inc. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. This material may not be copied or reprinted without the express written consent of ZimVie, Inc. ZV0930 REV A 04/23. ©2023 ZimVie, Inc. All rights reserved.

