



Work Smarter.

Smart Denture Conversions

with Separable Fastener Technology



Smart Denture Conversions

Immediate full arch provisionalization


Convert a stronger provisional in less time

Smart Denture Conversions eliminates the need for large holes in the denture by allowing a closed-tray pickup. This is possible by fastening the Ti-Base to a multi-unit abutment with a unique prosthetic screw, the Separable Fastener.

The Separable Fastener divides into two pieces for the pickup and the denture can be easily removed from the mouth prior to major modifications.


Faster

Process dentures in 30 minutes vs. the traditional two hours, significantly reducing the overall treatment time.

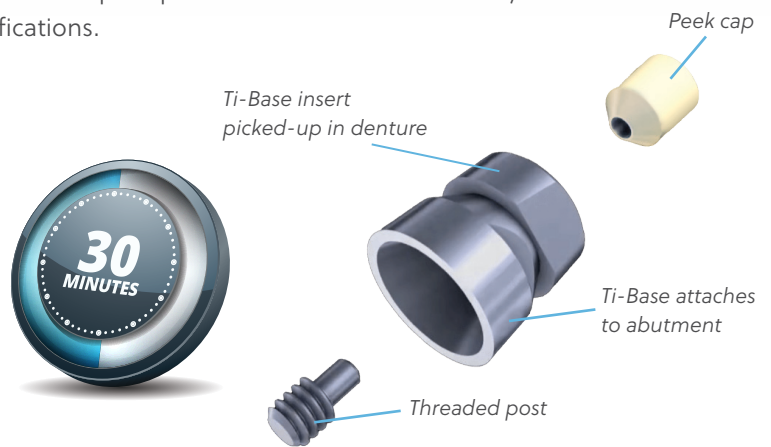


Stronger

Smaller pilot holes help reduce fractures by preserving the structural integrity of the prosthesis. Large holes remove more material which can weaken the prosthesis leading to a higher rate of fracture.

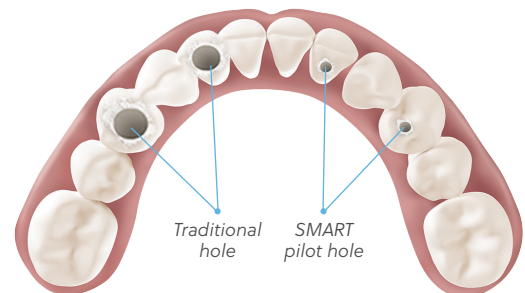

Better

Create better contours without the need to unnecessarily bulk the prosthesis. Improve occlusion by utilizing a fully occluded pickup technique, reducing the need for major occlusal adjustments.

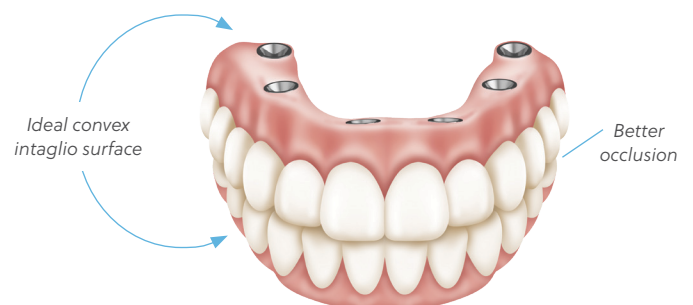

Chairtime is a valuable resource...

Work Smarter!

Conventional Conversions

2+ hours to process

Significant fracture rate

Damage to occlusal surfaces

Poor aesthetics

Less flexibility for surgical and restorative team



Smart Denture Conversions

Decreased chair time to 30 minutes

Improves productivity

Improves profitability

Stronger temporary acrylic bridge

Decreased fracture rate

Contact your local ZimVie Sales Representative about our Full Arch Solutions.

smiletoday® Full Arch Smart Denture Conversions



Ordering Information


Smart Denture Conversions Kits



Premium Starter Kit





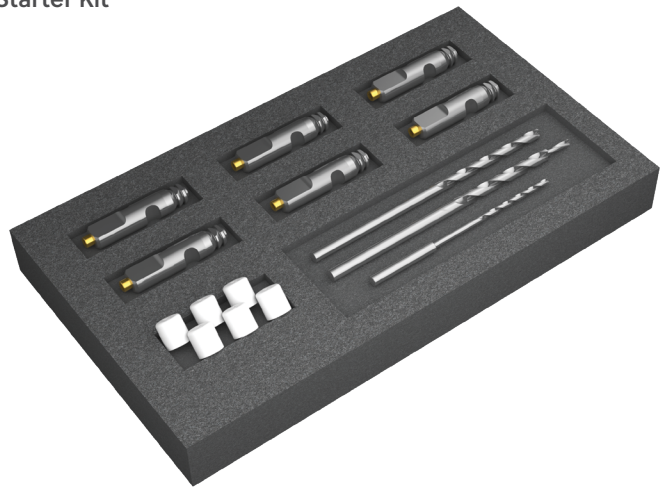


Recharge Kit w/Tall Spare Parts




Recharge Kit w/Press-On Caps

Implant System	Description	Item No.
TSX®, TSV®, and Eztetic® Implants	Premium Starter Kit	ZVTS-PSK
	Recharge Kit w/Tall Spare Parts	ZVTS-RK
	Recharge Kit w/Press-On Caps	ZVTS-RKPOC

Implant System	Description	Item No.
T3® PRO, T3®, and Certain® Implants	Premium Starter Kit	ZVLP-PSK
	Recharge Kit w/Tall Spare Parts	ZVLP-RK
	Recharge Kit w/Press-On Caps	ZVLP-RKPOC



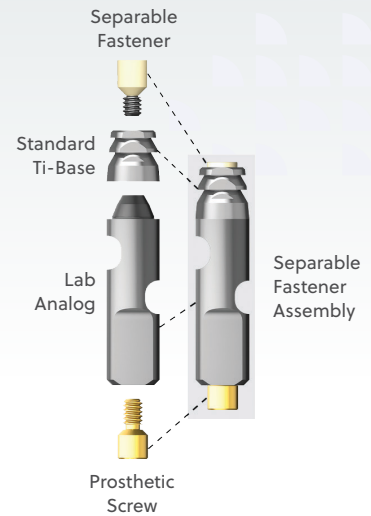


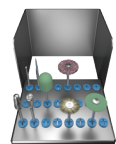
SMART DENTURE CONVERSIONS COMPONENTS		TSX®, TSV®, and Eztetic® Implants	T3° PRO, T3°, and Certain® Implants
Product	Description	Item No.	Item No.
	Separable Fastener Assembly, 10 Pack (Includes Drill Kit, ZV-DK)	ZVTS-SFA10PKDK	ZVLP-SFA10PKDK
	Standard Ti-Base, 10 Pack 4.6 mmH (Includes Prosthetic Screw)	ZVTS-STB10PK	ZVLP-STB10PK
	Tall Ti-Base, 10 Pack, 6.1 mmH (Includes Prosthetic Screw)	ZVTS-TTB10PK	ZVLP-TTB10PK
	Universal Multi-Unit Abutment Screw, 0.050 Hex	UMUA-S	-
	Gold-Tite® Retaining Screw, 0.048 Hex	-	LPCGSH



SMART DENTURE CONVERSIONS COMPONENTS

Product	Description	Item No.
	Separable Fastener, 10 Pack	ZV-SF10PK
	Tall Separable Fastener, 10 Pack	ZV-TSF10PK
	Lab Analogs, 10 Pack	ZV-LA10PK
	Press-On Cap, 6 Pack Press-On Cap, 10 Pack	ZV-POC6PK ZV-POC10PK
	Protective Plug, 6 Pack Protective Plug, 10 Pack	ZV-PP6PK ZV-PP10P
	Coating Mandrel, 10 Pack	ZV-CM10PK

SMART DENTURE CONVERSIONS TOOLS

Product	Description	Item No.
	Drill Kit	ZV-DK
	Individually packed drills: Pilot Drill, 1-Pack Access Drill, 1-Pack Cleanout Drill, 1-Pack	ZV-PD1PK ZV-AD ZV-COD
	Pin Vise, 1-Pack	ZV-PV
	Low Torque Driver	ZV-LTD
	Retrieval Tool	ZV-RT
	Bur Block Kit	ZV-BBK

For more information, visit [ZimVie.com](https://www.zimvie.com)

ZimVie

4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Phone: +1-561-776-6700
Fax: +1-561-776-1272



Unless otherwise indicated, as referenced herein, all trademarks are the property of ZimVie; and all products are manufactured by Smart Denture Conversions and marketed and distributed by ZimVie Dental and its authorized marketing partners. Universal Multi-Unit Abutments are manufactured by Terrats Medical, LLC. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie. ZV1699 REV A 11/23 ©2023 ZimVie. All rights reserved.